



## Mini Implant Drill Guide Technique Guide

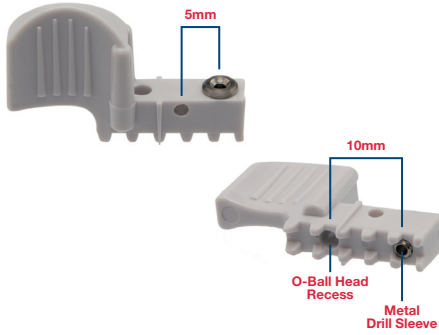


1. Make central mark on crest.



2. Place dental floss into side opening and align floss over central mark.

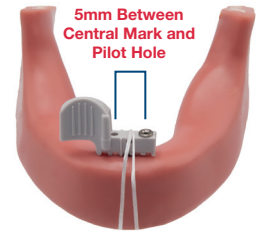

3. Drill a 6mm deep pilot hole through metal ring with 1.3mm Salvin Power Point<sup>®</sup> Pilot Drill after sounding site with periodontal probe.

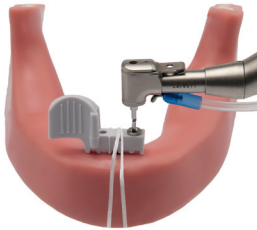


4. Insert first Mini Dental Implant.




5. Place Salvin Mini Implant Drill Guide O-Ball Head Recess at base of handle over exposed O-Ball head.




6. With Salvin Mini Implant Drill Guide in place over exposed O-Ball of Mini Dental Implant #1, drill pilot starter hole for Mini Dental Implant #2.

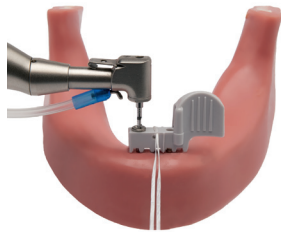

7. Place second Mini Dental Implant.

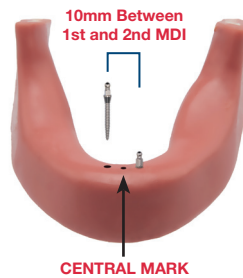


8. Reverse Salvin Mini Implant Drill Guide and place receptor hole over Mini Dental Implant #1 O-Ball top and drill #3 starter hole.

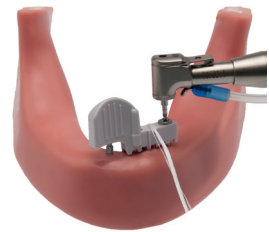

9. Insert #3 Mini Dental Implant in pilot hole.



10. Place receptor hole of Salvin Mini Implant Drill Guide over O-Ball Top of Mini Dental Implant #2 and drill #4 pilot hole.

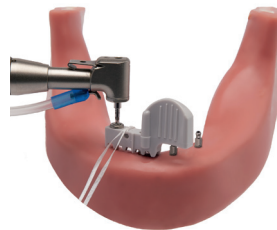

11. Insert #4 Mini Dental Implant in pilot hole.

